
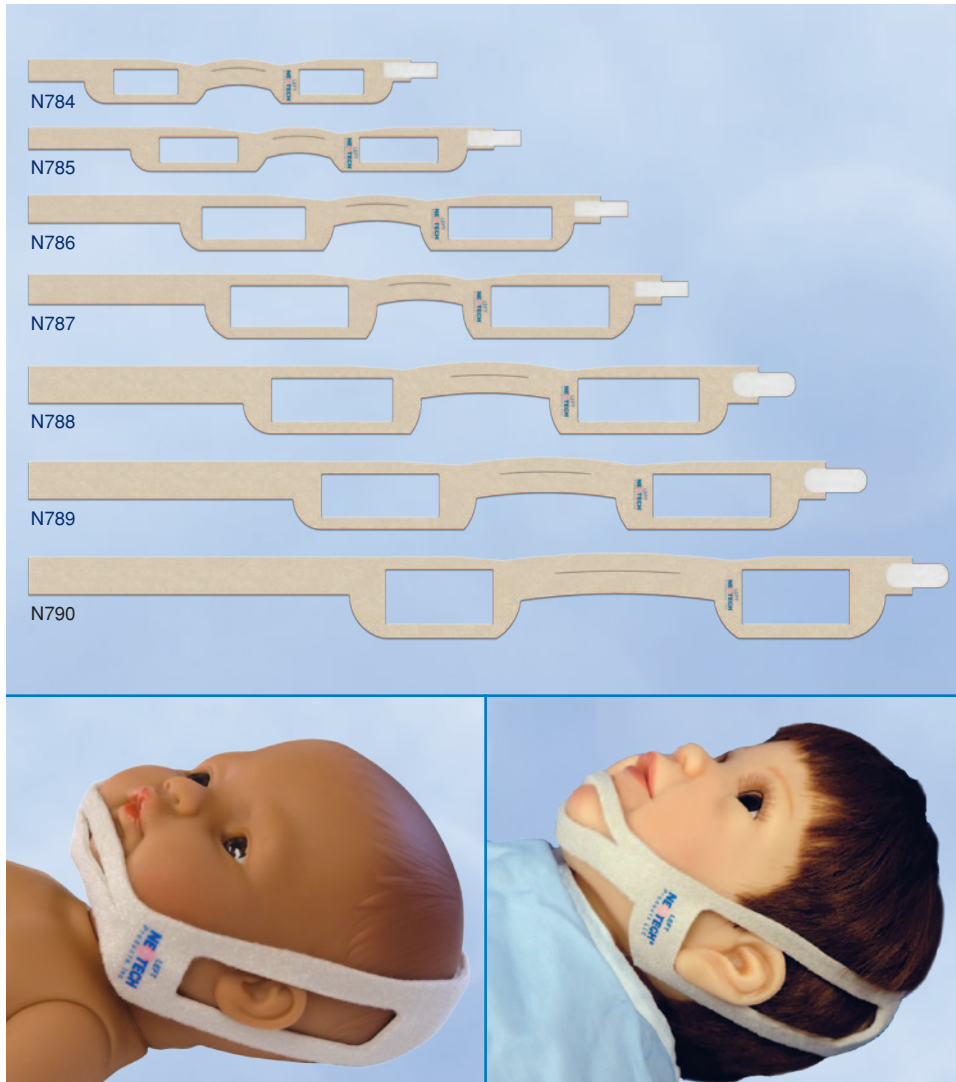


## Neotech ChinStrap<sup>™</sup>

Respiratory Support Accessory

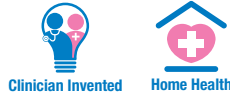

### FEATURES AND BENEFITS

- Soft, breathable, skin friendly NeoFoam<sup>®</sup> wicks perspiration
- Flexibility enables minimal mouth movement
- Form fitting and adjustable
- Split chin design for comfort and stability
- Ear openings to prevent folding over
- Designed to reduce the risk of pressure ulcers or necrosis
- Unique material helps to prevent sliding
- Not made with natural rubber latex or plasticizer DEHP
- Individually packaged

Cat No.	Item	Qty/Unit
N784	Nano	20/box
N785	Micro	20/box
N786	Small	20/box
N787	Large	20/box
N788	X-Large	20/box
N789	Macro	20/box
N790	Jumbo	20/box



Made in USA

U.S. Patent #10,342,693

# Neotech ChinStrap™

## Respiratory Support Accessory

### INDICATIONS FOR USE

The Neotech ChinStrap is intended to help keep a patient's mouth closed during respiratory therapy. It is intended for use on pediatric (neonates, infants and children) patients.

### DIRECTIONS FOR USE

#### Step 1

Select proper size by measuring circumference from center of chin, over the ear, to posterior of head.



#### Step 2

Place Neotech ChinStrap across chin with indentation facing neck and smooth, white side against skin.



#### Step 3

Position wider, lower portion of split chin strap under chin and upper portion across front of chin.

#### Step 4

Pull strap toward posterior of head with ears positioned inside ear openings and gently fasten hook tab to strap. Side marked "LEFT" should be under patient's left ear.



Note: May be trimmed for proper fit. Ensure that strap is not too tight or pressing on ears. Hook tab should completely overlap strap.



#### Step 5

Replace every 5 days or per hospital protocol, whichever is sooner.

Replace immediately if worn or visibly soiled.

### GENERAL INFORMATION

#### Product Materials

NeoFoam, hook fabric  
Made with natural rubber latex: No  
Made with plasticizer DEHP: No  
Consult MSDS for complete material information.

#### Size

N784 Nano: fits head circumference 22-27 cm  
N785 Micro: fits head circumference 28-35 cm  
N786 Small: fits head circumference 33-42 cm  
N787 Large: fits head circumference 36-46 cm  
N788 X-Large: fits head circumference 38-53 cm  
N789 Macro: fits head circumference 39-58 cm  
N790 Jumbo: fits head circumference 40-66 cm

#### Packaging

Individually packaged: Yes  
Package type: Polyethylene bag  
Dispenser box: Yes (N784-N787)

Visit [neotechproducts.com](http://neotechproducts.com) for the most up-to-date information.

Boxes are printed with environmentally friendly soy ink on recyclable material.

### CAUTIONS & WARNINGS

The Neotech ChinStrap is not intended for use on adults.

Federal law restricts this device to sale by or on the order of a physician.

Discontinue immediately if skin irritation occurs.

Single patient use only.



For more information or to order your FREE sample call  
**1-800-966-0500** or visit [neotechproducts.com](http://neotechproducts.com)

Neotech Products LLC • 28430 Witherspoon Pkwy, Valencia, CA 91355