

NeoGrip[®]
 Tubing and Cable Holder



FEATURES AND BENEFITS

- Helps to keep tubes, cables and circuits together
- Allows for better patient mobility
- Attaches to linen or clothing without damaging
- Helpful during kangaroo care
- Open and close repeatedly with no residue
- Can be used in clinical or home care setting
- Great for ambulatory patients
- Not made with natural rubber latex or plasticizer DEHP

<i>Cat No.</i>	<i>Color</i>	<i>Qty/Unit</i>
N601	Teal (VersaGrip)	40/box
N602	Pink/Blue	40/box
N602B	Blue	40/box
N602P	Pink	40/box
N602R	Red	40/box
N602G	Green	40/box
N602Y	Yellow	40/box
N602O	Orange	40/box
N602V	Violet	40/box
N603	White	40/Box



Made in USA



Clinician Invented

Patent #D814,568

NeoGrip®

Tubing and Cable Holder

INDICATIONS FOR USE

The NeoGrip/VersaGrip is intended to organize tubes, cables and circuits.

DIRECTIONS FOR USE

Step 1

Separate hook and loop material.

Step 2

Insert tubes, cables and circuits.

Step 3

Align hook and loop and apply pressure to secure.

Step 4

To remove, slowly peel hook and loop apart and carefully remove tubes, cables, and circuits.

Note: Replace immediately if hook and loop is worn.

GENERAL INFORMATION

Product Materials

Hook and loop fabric, mylar, stainless steel

Made with natural rubber latex: No

Made with plasticizer DEHP: No

Size

NeoGrip

Closed length: 115 mm (not including clip)

Width: 25 mm

VersaGrip (N601)

Closed length: 150 mm (not including clip)

Width: 25 mm

Packaging

Dispenser box: Yes

Visit neotechproducts.com for the most up-to-date information.

CAUTIONS

Do not use if hook or loop fabric is worn.

Single patient use only.



For more information or to order your FREE sample call
1-800-966-0500 or visit neotechproducts.com

Neotech Products LLC • 28430 Witherspoon Parkway • Valencia, CA 91355