

## NeoSmile™ Plus

Temperature Probe Cover



### FEATURES AND BENEFITS

- Reflective foil helps protect against ambient and radiant heat
- Cute smiley face design
- Skin friendly silicone gel adhesive base
- Reposition as needed
- Foam layer insulation
- Probe pocket for disposable or reusable probes
- Not made with natural rubber latex or plasticizer DEHP

Cat No.	Size	Qty/Unit
N7320	1" Diameter	150/box

# NeoSmile™ Plus

## Temperature Probe Cover

### INDICATIONS FOR USE

---

The NeoSmile is intended to cover and secure a temperature probe.

### DIRECTIONS FOR USE

---

#### Preparation

Clean and dry skin per hospital protocol.

Note: It is important for skin to be as clean and dry as possible. NeoSmile will not adhere properly to moist skin or hair. Oils and lotions will also adversely affect adhesion.

DO NOT USE ALCOHOL.

#### Application

##### Step 1

Peel and discard liner. Place NeoSmile Plus over probe and skin.

##### Step 2

Reposition NeoSmile Plus as needed.

#### Removal

##### Step 1

Replace NeoSmile Plus every 3 days or per hospital protocol, whichever is sooner.

### GENERAL INFORMATION

---

#### Product Materials

FLEXmark® MM 200 gold foil, silicone adhesive, white polyethylene foam  
Not made with natural rubber latex  
Not made with plasticizer DEHP

#### Size

Diameter: 25 mm

#### Packaging

Standard box, no dispense feature

Perforated liner

*Visit [neotechproducts.com](http://neotechproducts.com) for the most up-to-date information.*

*Boxes are printed with environmentally friendly soy ink on recyclable material.*

### CAUTIONS & WARNINGS

---

Discontinue immediately if skin irritation occurs.

NeoSmile will not adhere properly to moist skin or hair. Oils and lotions will also adversely affect adhesion.

Do not use alcohol to clean skin prior to application.

Single patient use only.



For more information or to order your FREE sample call  
**1-800-966-0500** or visit [neotechproducts.com](http://neotechproducts.com)

Neotech Products LLC • 28430 Witherspoon Pkwy, Valencia, CA 91355