
bili-ruler™

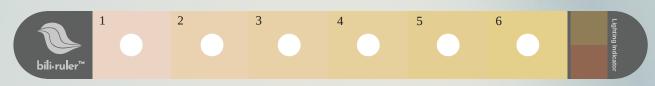

EASY TO USE GUIDE



- Place the infant in natural lighting conditions. The Lighting Indicator will appear uniform in color.
- Gently press each of the clear circles against the tip of the baby's nose to slightly blanch the skin. Compare to the surrounding color swatch.
- Determine the color swatch that matches best and note the number.
- Refer to the table on the accompanying case to determine the corresponding range of serum bilirubin levels and assess need for further diagnostic evaluation and treatment. (We recommend serum bilirubin testing for bili-rulerTM scores of 3 or greater.)