LifeStart™

EN Instructions for Use

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IT Manuale d'uso

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NO Bruksanvisning

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SE Bruksanvisning

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EE Kasutusjuhend

HR Upute za uporabu

sk Návod na používanie

Contents

LifeStart™

EN	Instructions for Use	1
CZ	Návod k použití	17
DE	Gebrauchsanweisung	33
ES	Instrucciones de uso	49
FR	Instructions d'utilisation	65
IT	Manuale d'uso	81
NL	Gebruiksaanwijzing	97
NO	Bruksanvisning	113
PL	Instrukcja użytkowania	129
SE	Bruksanvisning	145
TR	Kullanım Talimatları	161
DK	Brugervejledning	177
EE	Kasutusjuhend	187
HR	Upute za uporabu	197
SK	Návod na používanie	207

Neonatal Unheated Mattress for LifeStart™

EN	Instructions for Use	217
CZ	Návod k použití	218
DE	Gebrauchsanweisung	218
DK	Brugervejledning	219
EE	Kasutusjuhend	219
ES	Instrucciones de uso	220
FR	Instructions d'utilisation	220
HR	Upute za uporabu	221
IT	Manuale d'uso	221
NL	Gebruiksaanwijzing	222
NO	Bruksanvisning	222
PL	Instrukcja użytkowania	223
SE	Bruksanvisning	223
SK	Návod na používanie	224
TR	Kullanım Talimatları	224

LifeStart™

EN Instructions for Use

Contents

1.	Introduction	3
	General Description	3
	Description of Parts	3
2.	Symbols	4
3.	Safety	6
	Intended Use	6
	Indications for Use	6
	Contraindications	6
	Warnings	6
	Precautions	7
	Environmental Advice	7
4.	Setting Up	8
	Fitting Batteries (4 x AA type - batteries not supplied with LifeStart)	9
	Fitting the basket	9
5.	Accessories	9
	Recommended Configuration	10
	Gas Cylinders	11
	Drape	11
	Equipotential Earth Connection	11
6.	Operation of System	12
	Before Use	12
	Height Adjustment	12
	Wheels and Locking Mechanisms	12
	Timer Unit	13
	Before, During or After Use	13
7.	Infection Control, Cleaning and Maintenance Instructions	13
	Cleaning Guidelines	14
	Periodic Preventive Maintenance	14
8.	Troubleshooting	14
	Before Use	14
	Height Adjustment	14
	Wheels and Locking Mechanisms	14
	Timer Unit	14
9.	Technical Specification	16

1. Introduction

General Description

The LifeStart™ is a height adjustable platform that can be positioned by the mother's side at birth. A cylinder holder is available for users to add medical air and oxygen facilities.

A range of Inspiration Healthcare recommended third party accessories are available to fit onto the LifeStart with the addition of a basket to store necessary airway consumables.

Description of Parts

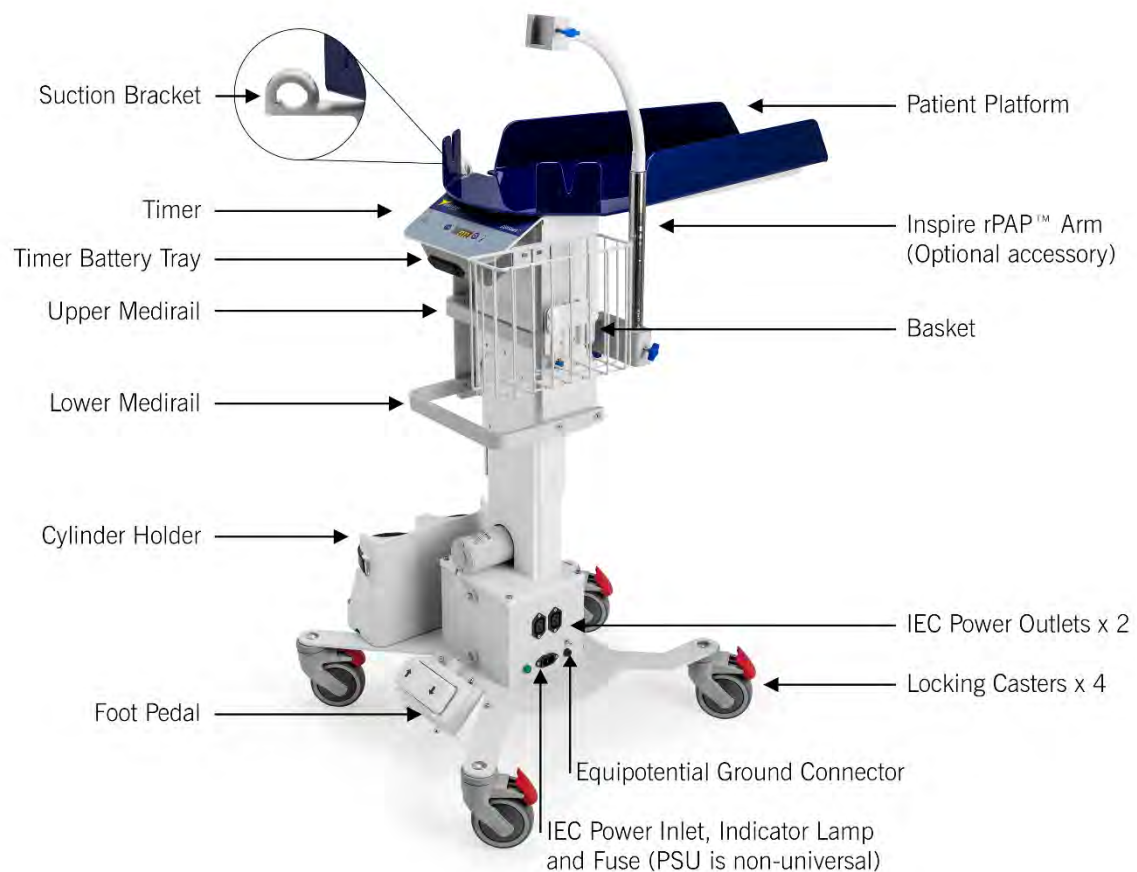
Figure 1.0: LifeStart™ Base Unit

Part Number	Part Description
LSU003	LifeStart™ (110V)
LSU003a	LifeStart™ (230V)
LSA011	Inspire rPAP™ Arm (Optional accessory)

2. Symbols

Battery charge status

Power to timer unit

00:00

Start/Stop timer

Raise platform height

Lower platform height

General warning sign

Within European Union the product must be disposed of in accordance with the WEEE Directive

Refer to instruction manual/booklet

Type B applied part

Warning: electricity

Keep dry

Keep away from sunlight

Date of manufacture

Manufacturer

Catalogue number

Batch code

RxOnly

Caution: Federal law restricts this device to sale by or on the order of a physician

Serial number

Equipotentiality

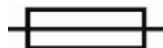
Do not use if package is damaged

Fragile, handle with care

Voltage

Fuse

Max. Patient Weight

Max. Accessory Weight

Unit Weight

Medical Device

Authorized representative in the European Community

Temperature Limit

Humidity Limitation

3. Safety

Intended Use

The LifeStart is a height adjustable patient platform that can be used to facilitate neonatal transition at birth. It provides a stable platform for mother side resuscitation and stabilization practices.

It can be used in hospital delivery rooms, operating theatres, newborn nurseries and neonatal intensive care units.

Indications for Use

Infants requiring resuscitation/stabilization at birth.

Contraindications

The LifeStart is not suitable for patient transport.

The LifeStart must not be used on patients over 10 kg.

Warnings

The LifeStart is not to be used for patient transport.

The LifeStart must not be used on patients over 10 kg.

The LifeStart is rated to a degree of Ingress Protection (IP) 30.

Never leave an infant unattended on the LifeStart patient platform.

Ensure the LifeStart patient platform is set to the correct height prior to birth.

Do not adjust the height of the patient platform whilst the patient is on the LifeStart.

All LifeStart height adjustments must be completely free from obstructions and the patient platform should be clear of the bed or operating table. Failure to observe due care could pose a risk to the neonate, mother, user or other medical staff. In addition, it could risk instability or cause damage to the product, bed or operating table.

Do not position the LifeStart so that it is difficult to disconnect the device from the mains electricity supply.

Ensure any damage is checked prior to use. If there is any sign of damage, remove the LifeStart from service and report to your local distributor. The LifeStart can only be repaired by an Inspiration Healthcare trained personnel.

This product should not be modified without prior approval from an Inspiration Healthcare representative.

The total weight of all accessories on the LifeStart should not exceed 15 kg.

Connecting electrical equipment to the Multiple Socket-Outlets (IEC Power Outlets) effectively leads to creating a Medical Electrical system and the result can be a reduced level of safety.

Maximum output from the IEC sockets should not exceed 6.3 A.

To avoid the risk of electric shock, the LifeStart must only be connected to a supply mains with protective earth.

If at any stage (before, during or after use) the LifeStart is required to be switched off, where applicable turn off the electricity supply at the wall switch and unplug the power cable from the mains electricity supply.

Disconnect the power from the mains before cleaning the LifeStart.

Do not use the system in the presence of flammable anesthetic gases or oxygen enriched atmospheres as these may pose an explosion hazard.

The user must avoid or minimize reciprocal interference posed by the presence of the LifeStart during use.

The user must avoid or minimize electromagnetic, high frequency and other interference between the LifeStart and any other devices.

Remove batteries when storing the product for a long period of time.

Precautions

Refer to section 6.Operation of System to prevent damage to the product and ensure safety of the patient, users and other staff.

Refer to section 7.Infection Control, Cleaning and Maintenance Instructions to ensure safety.

Use only Inspiration Healthcare approved accessories, contact your local distributor for more information.

Care should be taken when attaching accessories to the Medirails to ensure that the accessories do not obstruct the LifeStart parts when the height is being adjusted, particularly when the device is in the lowest position.

Ensure the caster lock mechanisms are applied when adjusting the LifeStart height or treating patients.

Ensure the LifeStart platform height is adjusted to the lowest position before inserting gas cylinders into the Cylinder Holder.

Maximum cylinder size for medical air and oxygen is D (universal pin index valve).

Ensure the mains power cable and gas hoses do not result in a trip hazard.

User should perform independent electrical safety testing upon receipt of the LifeStart, prior to putting it into operation.

Environmental Advice

This device is marked with the WEEE symbol (crossed-out wheeled bin) to indicate that it is electronic equipment covered by the Directive 2002/96/EC on waste electrical and electronic equipment. This is a European directive but applies worldwide. In European countries the WEEE symbol reminds you that all electrical and electronic products must be taken to a separate collection at the end of their working life. Do not dispose of this product in normal domestic or commercial waste - contact your local authority for advice on disposal.

4. Setting Up

Refer to the illustration in Description of Parts in section 1.Introduction.

Before operation, remove the packaging and check the rating label displays the correct power supply voltage.

Connect the power cable to the IEC power inlet. The power cable can be secured to the base of the unit using the designated 'P' clip provided.

Plug the power cable to the mains electricity supply and where applicable turn on the electricity supply at the wall switch.

The batteries for the Timer will not be supplied with the LifeStart. Please refer to Fitting Batteries section below on how to fit the batteries.

User should perform independent electrical safety testing upon receipt of the LifeStart, prior to putting it into operation.

Operate lock mechanisms on the casters to secure the device.

Prior to use, all product functions should be checked.

When positioning the LifeStart, please ensure that it is not positioned in such a way that it makes it difficult to disconnect the device from the mains electricity supply.

Important Notes

To comply with safety regulations, the unit should only be connected to the mains with an earth connection using the cable supplied by Inspiration Healthcare.

Two additional mains power IEC sockets are provided on the side of the LifeStart to connect other equipment. To connect other equipment use an IEC power supply mains cable. Maximum output from the IEC sockets should not exceed 6.3 A.

Fitting Batteries (4 x AA type - batteries not supplied with LifeStart)

To fit or replace the batteries in the Timer, remove the Timer Battery Tray underneath the Timer by pushing the tabs at each side inward and then sliding out from the Timer. Fit 4 x AA type batteries, observing correct polarity as indicated inside the Battery Tray. Please change all four batteries at the same time to avoid uneven discharge and possible damage caused by leaking batteries. Replace the Timer Battery Tray by pushing it in until it clicks and is therefore fully engaged.

Fitting the basket

LSU003 and LSU003a series are both supplied with a basket. To fit the basket, slide the fitted bracket on the basket onto the Medirail. Secure the basket using the Medirail thumb screw found at the bottom of the bracket.

5. Accessories

Medirails are fitted to the LifeStart to enable the system to carry a range of medical equipment in close proximity to the patient.

Attach the required accessories to the Medirails, ensuring that they are fully secure and do not present an obstruction to other items or to the power outlets. The user must also ensure when attaching the accessories that they are not obstructing the ability to connect/disconnect the device. It is anticipated that the following accessories might be required to provide necessary treatment and support for the patient: gas cylinders, neonatal warming system, suction system suitable for a newborn infant, air/oxygen blender, resuscitator (Inspire rPAP™). A recommended configuration is illustrated below in Recommended Configuration.

Recommended Configuration

The image shows the LifeStart fitted with the accessories in the recommended configuration.

All LifeStart units are fully height-adjustable. Consideration should be given to this when fitting the accessories.

Ensure that any equipment optionally fitted to the LifeStart is properly secured using an appropriate clamp or rubber bungee.

Important Notes

The total weight of all the accessories should not exceed 15 kg and should be distributed and mounted evenly to avoid destabilising the unit.

Care should be taken when attaching accessories to the Medirails to ensure that the accessories do not obstruct the LifeStart parts when the height is being adjusted, particularly when the device is in the lowest position.

Refer to the manufacturer's instructions for any accessory used in conjunction with the LifeStart.

Please contact Inspiration Healthcare for a range of compatible devices available.

No attempt should be made to remove any components or sub-assemblies except by a suitably trained and qualified Inspiration Healthcare representative.

Gas Cylinders

Secure gas cylinders in place by attaching the rubber bungee to the cylinder holder using the supplied metal clips.

Drape

A sterile drape is available for use in the operating theatre and delivery suite to preserve the sterile field and prevent ingress of fluids. To apply the drape, ensure the plastic is carefully lifted over the patient platform and then over the LifeStart, covering all accessories attached. The soft nonwoven pad should be positioned over the patient platform.

Equipotential Earth Connection

An Equipotential Earth Connection on the side of the unit is necessary for electrical testing, which may be required under local regulations. Please consult local regulations to ascertain if this is required.

6. Operation of System

Important Notes

Check that there is no damage to the LifeStart unit or the power cable and ensure that the Medirails and the patient platform are securely attached.

Always apply the brakes before adjusting the height.

Do not use LifeStart if the brakes do not function correctly.

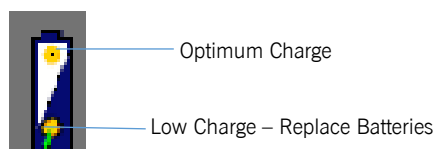
Ensure the LifeStart patient platform is set to the correct height prior to birth.

Do not adjust the height of the patient platform whilst the patient is on the LifeStart.

All LifeStart height adjustments must be completely free from obstructions and the patient platform should be clear of the bed or operating table. Failure to observe due care could pose a risk to the neonate, mother, user or other medical staff. In addition, it could risk instability or cause damage to the product, bed or operating table.

Before Use

Turn on the Timer and check the battery charge status indicator; replace the batteries as instructed in section 4.Setting Up if the low battery power indicator is illuminated. Please see illustration below.

Height Adjustment

All LifeStart units have electrical height adjustment, operated by the foot pedals.

To raise the patient platform, depress the pedal marked with the upward-pointing arrow; to lower the LifeStart, depress the pedal marked with the downward-pointing arrow.

When the platform reaches the desired height, release the pedal to stop the motor. When the platform reaches its highest or lowest position, the motor will automatically stop and the pedal should be released.

Wheels and Locking Mechanisms

All casters are designed to swivel. All four wheels are fitted with a lock mechanism. To engage, press down on the front of the locking tab until it clicks into the brake position; to release locking tab, press down the rear of the locking tab unit it clicks into unlocked position.

Timer Unit

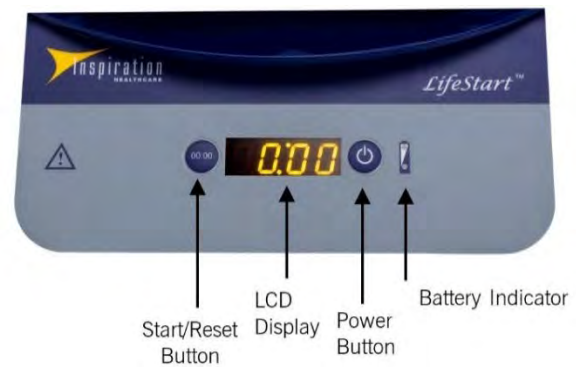
To turn on the Timer, press and hold the Power button for 3 seconds until the display illuminates showing “----”.

To start the Timer press and hold the Start/Reset button and the “-” symbols will progressively disappear from the right. When the last “-” symbol disappears, the timer will show 00:00 and will then start counting up, at which point the user can release the Start/Reset button. The display indicates the time in minutes and seconds.

The Timer indicates how much time has passed with the umbilical cord of the newborn baby still intact. It will emit single, double and triple beeps to indicate one, two and three minutes respectively without having to visually check the display.

To reset the Timer, press and hold the Start/Reset button for 2 seconds. This will reset the Timer to begin counting from 00:00.

To turn off the Timer and preserve battery life, press and hold the Power button for 2 seconds. The display will go blank when turned off. After the timer has been running for 20 minutes it will turn off automatically to preserve battery life.

Before, During or After Use

If at any stage the LifeStart is required to be switched off, where applicable turn off the electricity supply at the wall switch and unplug the power cable from the mains electricity supply.

7. Infection Control, Cleaning and Maintenance Instructions

Important Notes

Infection control and routine cleaning should be carried out in accordance with the local Infection Control Policy.

Products must be decontaminated prior to servicing or return to the supplier.

Any product being returned to Inspiration Healthcare or the local distributor must be accompanied by a decontamination certificate. A copy of the certificate should be affixed securely to the outside of the packaging.

All cleaning agents must be used in accordance with manufacturer's instructions for use.

Do not immerse any part of the LifeStart in fluids.

Do not use phenolic based products for cleaning.

Do not process LifeStart in an autoclave, steriliser or automatic washer-disinfector.

Do not use cleaning or decontamination methods different from those recommended without approval by Inspiration Healthcare.

Cleaning Guidelines

Ensure the LifeStart is disconnected from the mains electricity supply before cleaning.

It is recommended that after each use, all surfaces are cleaned using chlorine dioxide based disinfectant.

Clean the rubber bungee using a mild soap solution and soft cloth.

Rubber bungee can be replaced during periodic preventive maintenance.

Periodic Preventive Maintenance

Periodic preventive maintenance must be performed every 12 months by an Inspiration Healthcare trained personnel.

8. Troubleshooting

Before Use

All electrical items (cables, power inlets, power outlets, etc) should be checked for any sign of damage before each use of the system. If there is any sign of damage, or if any part of the electrical system is not working, remove the LifeStart from service and report to your local distributor. Check that the Medirails and the patient platform are securely attached.

Height Adjustment

At regular intervals, check that the height adjustment mechanism functions properly. The height adjustment stops automatically at the upper and lower limits of travel and should be smooth and quiet in operation. These limits are factory set and no attempt should be made to alter them.

If the height adjustment fails to function correctly, firstly check the electrical connection to the mains electricity supply. If this does not solve the problem, remove the LifeStart from service and report to your local distributor.

Manual height adjustment is not possible. The height adjustment mechanism should not be operated continuously for more than a 2 minute period. If this occurs the mechanism should not be operated again for 20 minutes to allow the system to cool down. Failure to observe this may lead to damage of the system.

Wheels and Locking Mechanisms

At regular intervals, inspect the wheels and brake mechanism to ensure they function correctly. Do not use the LifeStart if any of the wheels or brakes fail to function smoothly and effectively.

Timer Unit

If the battery indicator illuminates with a yellow light in the lower section of the symbol, then the batteries should be replaced as soon as convenient.

If the Timer fails to turn on, ensure the Power button is pressed for at least 3 seconds. If it still fails to power up, then replace the batteries as described in section 4. Setting Up.

If the Timer powers up but fails to start then ensure the Start/Reset button is pressed and held until all the “-“ symbols have disappeared. If it still fails to start, remove the LifeStart from service and report to your local distributor.

9. Technical Specification

Construction:	Fabricated in mild steel, with powder coat paint finish, for durability and stability. Compact, low-profile wheelbase to allow easy and convenient positioning, close to mother, in all situations (natural birth, assisted delivery, caesarean section, etc) Antistatic wheels with brakes Electrically operated height adjustment with actuator for smooth adjustment and complete stability Ergonomic, unobtrusive platform to give easy access to infant
Adjustment:	Patient platform height range from 70 cm to 110 cm from floor level Foot pedal operated adjustment to leave hands free and facilitate aseptic techniques
Timer:	Built-in timer with bright, backlit display to monitor cord clamping time Audio alerts at 1, 2 & 3 minutes from delivery Battery operated using 4×AA size 1.5 V alkaline cells (not supplied)
Power Supply:	200 VA~ 50/60 Hz
Input Voltage:	230 V~ or 110 V~
Dimensions:	Footprint 65 cm × 64 cm Overall Height 79 cm (lowest position from floor level)
Weight:	30 kg
Maximum Patient Load:	10 kg
Maximum Additional Equipment Load:	15 kg
Product Features:	Two Medirail mounting rails Two mains power IEC outlets for Inspiration Healthcare approved accessories
Compliance:	The LifeStart has been tested and approved to comply with the following international standards: EN ISO 60601-1: 2006+A1:2013 (Edition 3.1) Medical electrical equipment: General requirements for basic safety and essential performance EN ISO 60601-1-2: 2007 Medical electrical equipment: General requirements for basic safety and essential performance – collateral standard: Electromagnetic compatibility – requirements and tests The LifeStart is rated to a degree of Ingress Protection (IP) 30
Environmental:	
Ambient Temperature (Operating):	15°C to 40°C (59°F to 104°F)
Ambient Temperature (Storage):	-10°C to 55°C (14°F to 131°F)
Relative Humidity:	30% to 70%

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Instructions for Use

Infection Control and Cleaning

Clean with care using a damp soapy cloth and/or disinfectant wipes (pH neutral or alkaline).

- Where higher level of decontamination is required, use Sodium Hypochlorite (bleach) solution or similar (up to 10,000 ppm available Chlorine). After cleaning with bleach solution, wipe down to remove residual cleaning agent and dry.
- Do not immerse in fluids.
- Do not process in an autoclave, steriliser or automatic washer-disinfector.
- Do not use phenolic based products, abrasives or harsh chemicals for cleaning.

Disposal

Dispose of in line with local protocol.

Safety

Contraindications

Avoid direct contact with mattress in circumstances where patient’s skin is compromised.

Warnings


The user should carefully read the instructions for use before using the equipment.

This mattress should only be used with LifeStart™.

Do not use if there is any sign of damage. Ensure the surface integrity of the mattress and replace if necessary.

This mattress is not sterile. Where necessary the user should take appropriate precautions to protect the sterile field.

Part Number	Part Description
IHC-LSA060	Neonatal Unheated Mattress for LifeStart™

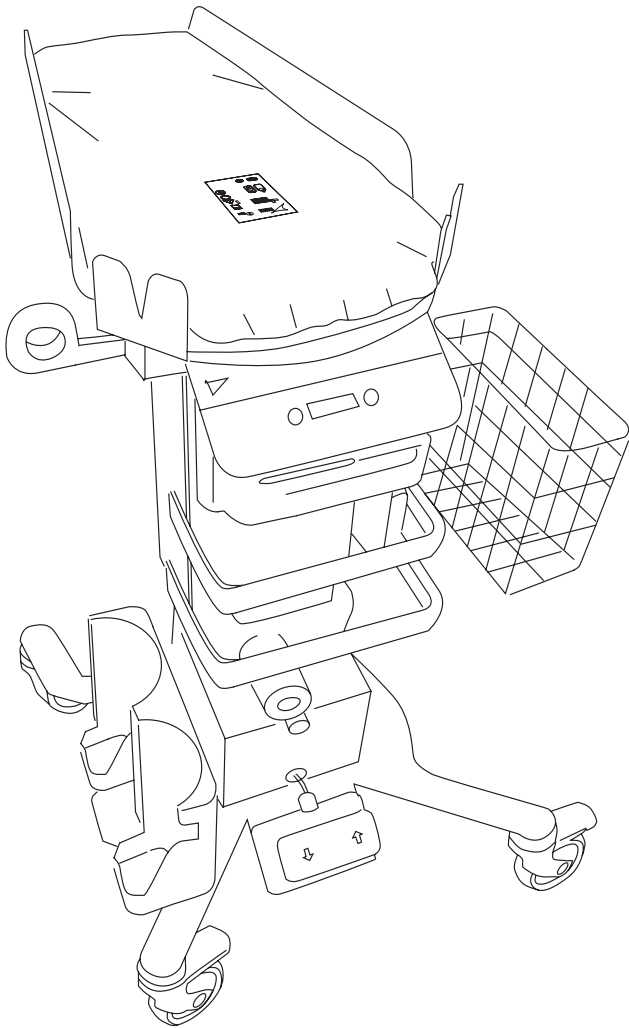	Consult Instructions for Use		Catalogue number		Do not use if package is damaged
	This way up. The mattress must be positioned with this symbol on top		Manufacturer		Keep dry
	The mattress may become damaged or perforated by sharp objects or instruments		Date of manufacture		Keep away from sunlight
			Serial number		Caution: Federal law restricts this device to sale by or on the order of a physician