

TarryGels™

Not made with natural latex rubber. DEHP and BPA free.

More information is available at www.dandlelionmedical.com.



INTENDED USE:

TarryGels provide gentle cushioning when a supportive pillow or surface is needed for hospitalized infants and may help to reduce the need for frequent re-positioning. The pillows are made from a soft polyethylene film filled with a non-toxic, water-based food gel that is resistant to bacterial and fungal growth. Available in 3 sizes: 3" x 6", 6" x 6", and 6" x 9". Each TarryGel comes with a disposable, non-woven cover.

WARNINGS / CAUTIONS:

- For hospital use only.
- Do not lie baby face down on gel in a way that could restrict baby's airway. Use with medical supervision.
- If baby requires thermal support, prewarm the gel in an incubator or warmer. Do not microwave as hot spots may form.
- Always use with cover.
- Do NOT X-RAY through gel as air pockets may distort x-ray results.
- Single patient use



CLEANING INSTRUCTIONS:

- Gel Pillow: To prevent cross-contamination between patients, gel pillows are indicated for single patient use. Any reuse of the pillows is not warranted by the manufacturer and should be made only after an appropriate method of sanitizing/disinfecting has been validated by the user.

INSTRUCTIONS FOR USE

- Prewarm the Gel Pillow in the incubator or under hot running water if necessary. Gel will maintain the ambient temperature of the environment. Place the pillow where needed. For a fuller pillow, push the gel to one side, and fold part of the bag underneath to achieve desired fullness.
- After extended use in a warm environment, the TarryGels may start to puff up. The gel will become watery and the bag will become puffy. Please discard the TarryGel when this occurs.

STORAGE:

Store below 75°F (24°C).

DISPOSAL:

No special requirements.

Please contact Dandle•LION Medical with any questions or comments
or to report any issue with the use of our product.

Rev 8:0925



Tarry Medical Products Inc.
dba Dandle•LION Medical
www.dandleLIONmedical.com

22 Shelter Rock Lane
203-791-9001

Danbury, CT 06810
© DandleLION Medical 2021