# Neotech Ninni™

## Breast-like Pacifier

**REF** Catalog No.  
N8701

Manufactured by  
NEOTECH PRODUCTS LLC  
28430 Witherspoon Parkway  
Valencia, CA 91355, USA  
1-800-966-0500  
neotechproducts.com

**Made in USA**

D526 Rev A 2025.11.17

English

Neotech Ninni™ Breast-like Pacifier  
INTENDED USE

The Neotech Ninni is intended to soothe and support non-nutritive sucking for newborns 0-6 months.

### DIRECTIONS FOR USE

#### Preparation

Step 1 Remove Neotech Ninni from packaging.

Step 2 Wash thoroughly with warm water and mild soap.

Note: Do not use a brush or other abrasives tools as they may degrade the material.

Step 3 Sterilize using an approved steam sterilization bag.

Note: Follow the bag manufacturer's instructions.

Repeat steps 2 and 3 after each use.

Store in a clean, dry container. Do not expose to direct sunlight for prolonged periods.

#### Inspection

Step 1 Visually and manually inspect the Ninni before each use.

Step 2 Pull the bulb portion of the nipple in all directions.

Step 3 Discard if any of the following are observed: cracks, tears, stickiness, discoloration, thinning of material, or changes in texture.

#### Patient Use

Step 1 Place Ninni in infant's mouth and ensure that lips flange around base.

Step 2 Place finger in the nipple to help guide the tongue.

Note: if infant doesn't accept the Ninni immediately, reinsert finger and gently press upward against the palate to encourage a latch-like sucking response.

### CAUTIONS & WARNINGS

Always use the Ninni with adult supervision.

Do not tie Ninni around baby's neck as it presents a strangulation hazard. Only use dedicated pacifier holders tested to EN 12586.

Not intended for chewing; do not use for teething babies.

Inspect the Ninni before each use. Discard immediately if damaged (cracks, tears, stickiness, discoloration, thinning of material, or changes in texture).

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