

# Quick Start Guide



This guide does not  
replace the User Manual

Intended for all the people  
authorized to adjust settings

English

## Turn ON the device

Press on



## Turn OFF the device




Press and hold



+ VALIDATE.

## Manage the treatment

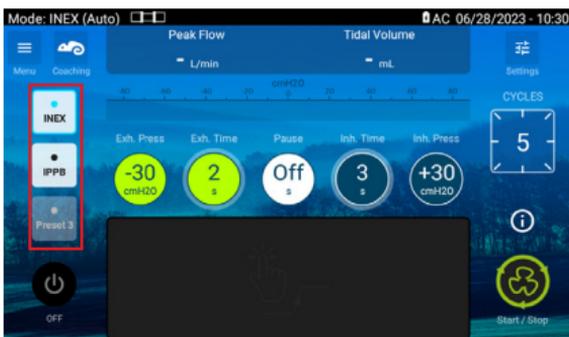

Select the preset

To start ventilation: press on




To stop ventilation: press and hold



+ VALIDATE.

**INEX manual mode:** slide your finger from right to left or left to right. To pause stop touching the pad.


## Unlock the Clinical Menu

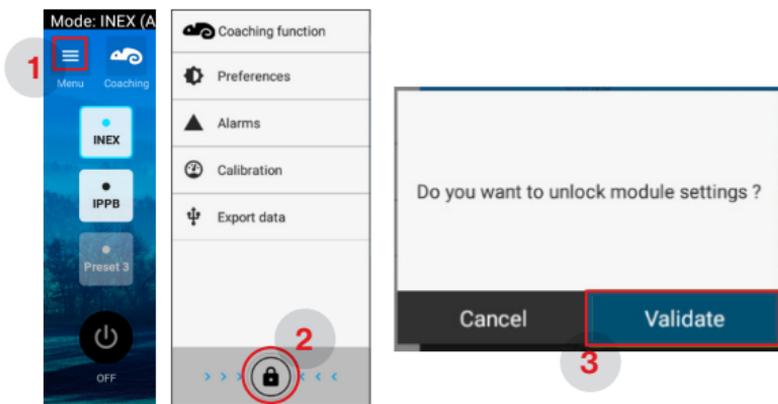

Press on the button **Menu** button (1), then **unlock Clinical Menu** pressing and holding (2) and **VALIDATE** (3).

## Setup a Treatment


Press on the **Settings** button, then, choose the **treatment mode**.

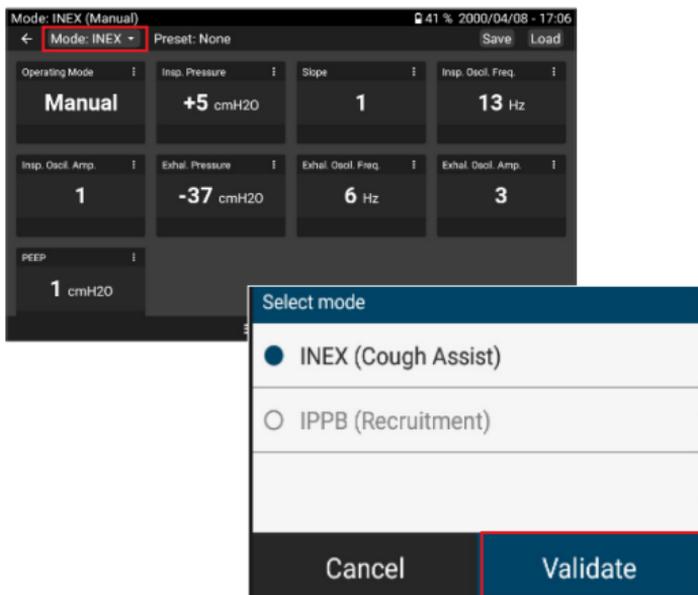

Adjust the settings and **Validate**.

## Save a preset

It is possible to store a preset by pressing the “**Save**” button on the bar on the top.

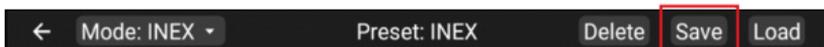

Choose **the preset, customize** the name and **VALIDATE**.


## Change the current parameters

**Press on the setting** you want to modify, **adjust** it, and **VALIDATE**.



## Perform a Calibration

Execute the **calibration** from the menu following the instructions. Then, validate and return to the home screen.

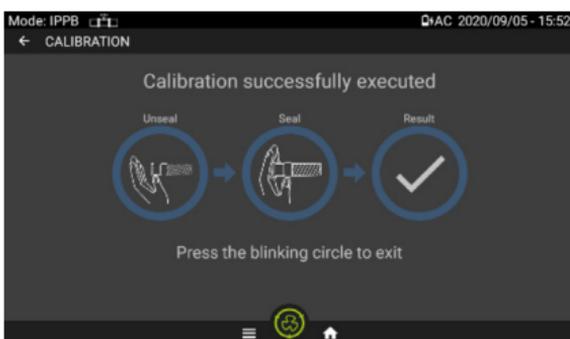

# Circuit configurations



Connect the breathing system to the patient circuit port.


**INEX Mode:** Simple limb circuit **without leak**



**IPPB Mode:** Simple limb circuit with **intentional leaks**



# Circuit assembly

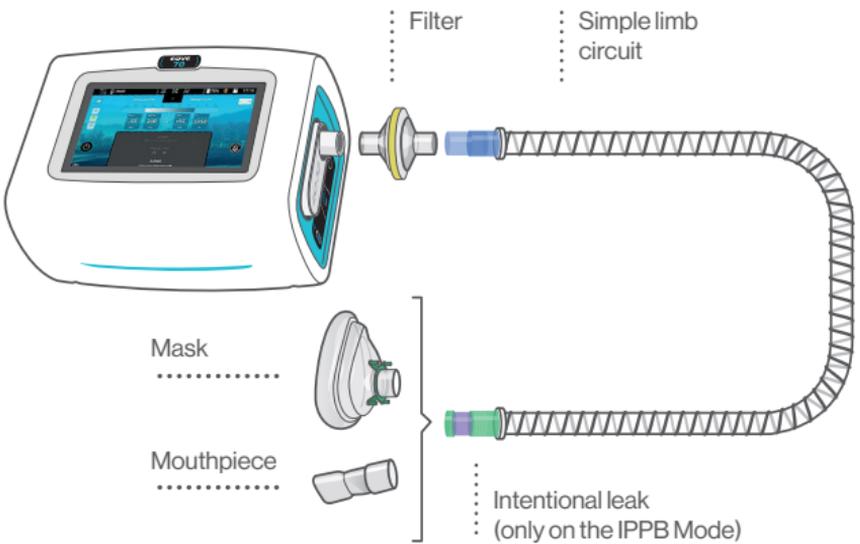

**Manufactured by**

EOVE  
Helioparc  
4, Boulevard Lucien Favre  
64000 Pau  
France  
[www.eove.fr](http://www.eove.fr)



Medical device class IIb - CE 0459 - Manufactured by EOVE. Please read the user manual.